Amplified Photo ID Telephone

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English Page 1

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CONTENTS

CONTENTS	1
INTRODUCTION	2
Unpacking the Telephone	3
DESCRIPTION	4
General Description	4
SETTINGS	6
Setting Up	6
Ringer Settings.....	6
Flash (Recall) Setting.....	6
Receiving Volume Setting	6
Speakerphone Volume Setting	7
USING THE TELEPHONE	8
Making a Call	8
Receiving a Call	9
Last Number Redial	9
Hearing Aid Use	9
Flash Signal	9
MEMORY	10
Storing a Number to Memory	10
Dialling a Number from Memory	10
TROUBLESHOOTING	11
SAFETY INFORMATION	12
DEVICES	13
GUARANTEE	14
RECYCLING DIRECTIVES	15

INTRODUCTION

!!! Please refer to our website: **www.geemarc.com** for an up to date user guide, as there may be important updates and changes you need to be aware of.!!!

Congratulations on purchasing your Geemarc PHOTOPHONE110. This is an amplified big button telephone The large button keypad is easy to see and use, it helps to prevent any misdialling. There are photo dial buttons, also designed to make dialling easier because you will be able to identify these speed dial buttons with pictures of your friends and family. This telephone is compatible with hearing aids.

Its is important that you read the instructions below in order to use your Geemarc telephone to its full potential. Keep this user guide in a safe place for future reference.

This guide explains how to use the following telephone:

PHOTOPHONE 110

INTRODUCTION

Unpacking the Telephone

When unpacking the telephone, you should find the following in the box:

- 1 PHOTOPHONE 110 unit
- 1 PHOTOPHONE 110 handset with curly cord
- 1 Telephone line cord
- 1 User Guide

For product support and help visit our website at
www.geemarc.com

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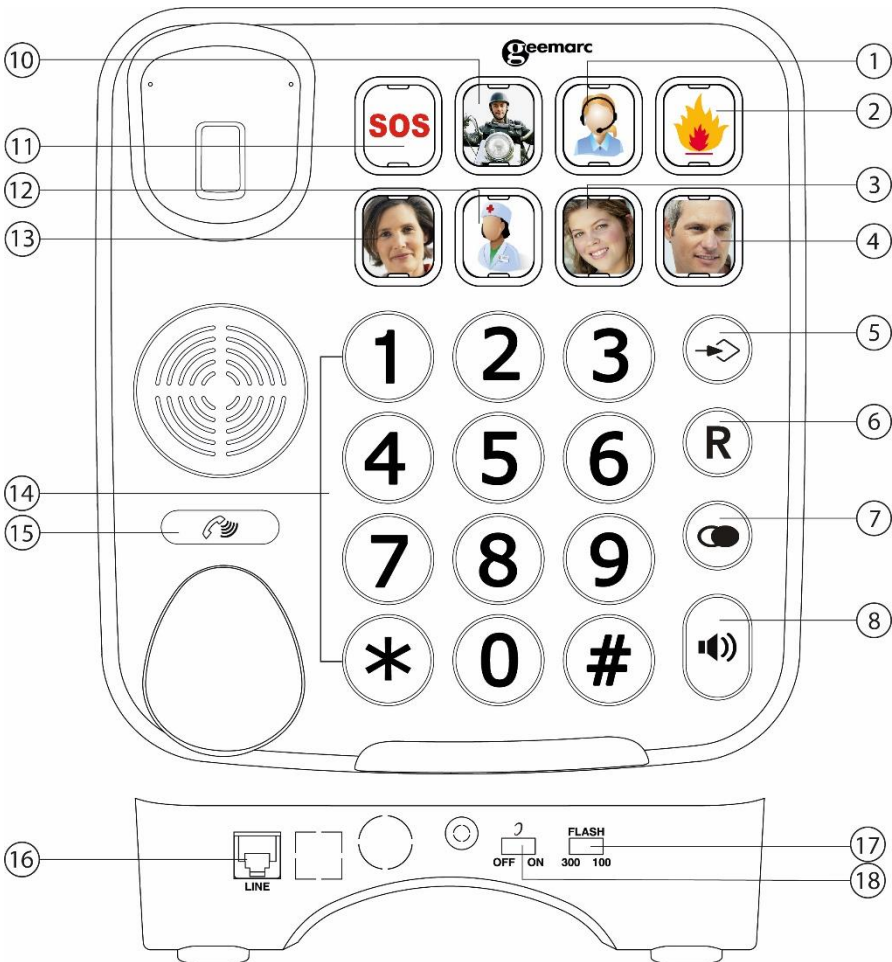
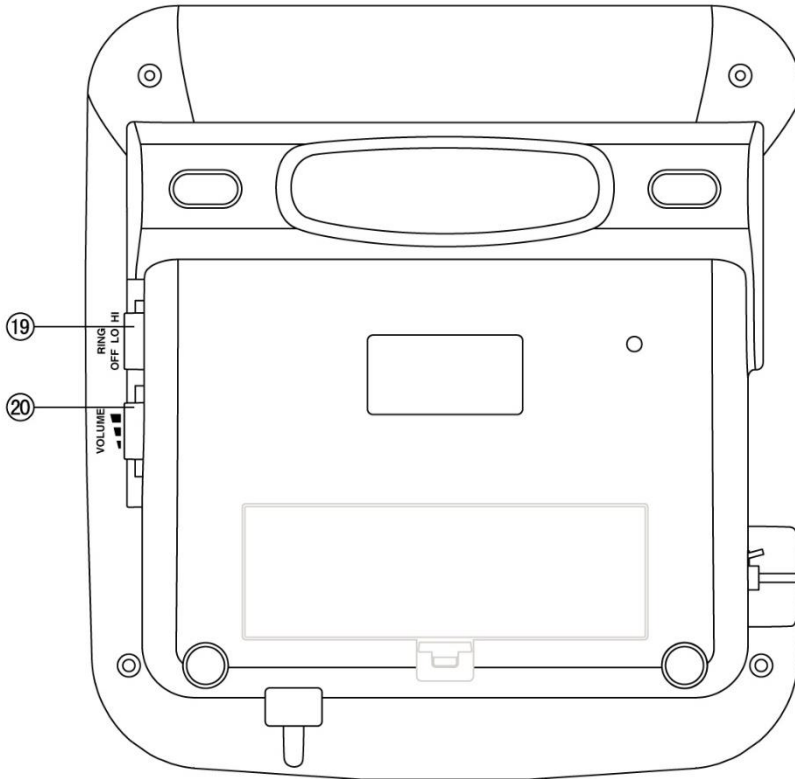
SETTINGS

Setting Up

Connect the handset to one end of the curly cord. Plug the other end of the curly cord into the jack on the left hand side of the PHOTOPHONE110 unit. Place the handset on the cradle.

Connect the line cord to the jack located at rear of telephone then connect the plug at the other end of the line cord into the wall socket or internet box*.

(*) Classified TNV-3 according to EN62368-1 standard

Ringer Settings

A switch located on the right side of the PHOTOPHONE110 unit allows the ringer volume to be switched to **HI** (maximum volume), **LO** (lowest volume) or **OFF** (if you would prefer not to be disturbed).

Flash (Recall) Setting

You can adjust the flash time by using the switch located on the rear side of your telephone phone. There are two levels available, 100ms and 300ms. Your telephone works with a flash time for United Kingdom and European (R=100ms), France (300ms). When the FLASH switch 100/300 is moved, your telephone's land line needs to stay unplugged for 10 minutes for the change to be processed.

Receiving Volume Setting

You can alter the receiving volume during a conversation according to your hearing requirements.

Use the Volume slide control  **VOL.** found on the right side of the telephone base to adjust the level. The volume adjustment available is 6-9 dB.

The **AMPLIFY** button  on the telephone base provides additional amplification of 18 dB i.e the volume adjustment will become 12-30B. When the amplify function is activated, the Amplify LED is lit.

SETTINGS

Setting a default for Amplification.

The Amplify **ON /OFF**  located at the rear allows the amplification facility to be switched **ON** or **OFF** as a default each time you use the telephone.

AMPLIFY ON/OFF Switch at ON Position

The additional amplification is automatically activated each time you use the telephone.

Pressing the Amplify key  during a call will turn off the additional amplification. However, you can still adjust the handset receive volume with the Volume slide bar.

Irrespective of how many times you press the amplify key  throughout a call, the next time you pick up the handset to make a new call, these functions will be activated once again.

AMPLIFY ON/OFF Switch at OFF Position

The additional amplification is deactivated each time you use the telephone.

Pressing the Amplify key  once during a call will enable additional amplification. If you press the amplify key  once more, the additional amplification feature will be turned off once again.

Irrespective of how many times you press the amplify key  throughout a call, the next time you pick up the handset, these functions will be turned off once again.

Speakerphone Volume Setting

In hands free mode you can adjust the speaker volume by using the volume controls  **VOL.** located on the right side of the phone.

USING THE TELEPHONE

Making a Call

Pick up handset.

When you hear the dialling tone, dial the number on the keypad.

To end the call simply replace the handset in the cradle

Making a Call – Hands Free

Press the  button to get a dialling tone

Dial the telephone number on the keypad.

To end the call, simply press  again

To activate the Hands Free function during the call, press the **SPEAKER** button  and replace the handset on the cradle at the same time.

To deactivate the Hands Free function during a call, lift the handset.

Note: When in hands free mode, only one person may talk at a time. The switch-over between speaker and microphone is automatic. This is dependent on the sound level of the incoming call and the microphone respectively. It is therefore essential that there are no loud noises, e.g. music, in the immediate vicinity of the telephone, as this will disrupt the hands free function.

USING THE TELEPHONE

Receiving a Call

When an incoming call is received, the telephone rings and the incoming call LED lights up.

Lift the handset and speak

To end the call simply replace the handset in the cradle

Receiving a call – Hands Free

Press  to speak hands free

To end the call press 

Last Number Redial

Lift the handset or press  for hands free use

Press  to redial the last number dialed.

Hearing Aid Use

This telephone is hearing aid compatible. Select the T mode on your hearing aid to enable this feature.

Flash Signal

The **FLASH** button can be used with special services such as Call Waiting (if provided by your service provider) or transferring external calls to another extension within a private switchboard.

MEMORY

Storing a Number to Memory

There are 8 memories pictures buttons (8 photo dial buttons). You can slip your contact's photograph under the perspex of the button. This is very useful if you tend to forget telephone numbers or names.

Left the handset on his base. Enter the telephone number you wish to store (up to 16 digits). If required, you can use the  button to insert a pause

Press  again.

Press the desired memory location (1 of the 8 photo dial buttons). The telephone number will be saved to that memory location.

Entering a new number into a memory location which currently stores a number will automatically erase the original number
Note: If you store family or friend numbers in photo key, you can put his or her photo in it.

Dialling a Number from Memory

Pick up the handset or no if you want call in speakerphone mode

Press the desired memory location (1 of the 8 photo dial buttons)

The telephone number will be dialled automatically

TROUBLESHOOTING

PHOTOPHONE110 does not ring

- Make sure that the telephone line cord is plugged in correctly and is not damaged.
- Connect another telephone to the telephone PTT socket or move your telephone to another socket to track if the failure is coming from your telephone or your PTT socket.
- There may be too many communication devices connected to a single telephone line e.g. another telephone, modem or facsimile machine. Contact your network provider for help in calculating the limit for your home or business line.
- Make sure the ringer setting is not switched to OFF.

No dial tone

- Make sure that the telephone line cord is plugged in correctly and is not damaged.
- Connect another telephone to the telephone PTT socket or move your telephone to another socket to track if the failure is coming from your telephone or your PTT socket.
- There may be too many communication devices connected to a single telephone line e.g. another telephone, modem or facsimile machine. Contact your network provider for help in calculating the limit for your home or business line.

One-touch memory key unuseful

- Check if you have stored number in it, if not, please store the number again.

SAFETY INFORMATION

General

When using your telephone equipment, the following basic safety precautions should always be followed to reduce the risk of fire, electric shock and injury to persons:

1. Read, understand and follow all warnings and instructions
2. Unplug the product from the wall outlet before cleaning. Do not use liquid cleaners or aerosol cleaners. Use a soft damp cloth for cleaning.
3. Do not place this product on unstable surfaces. The product may drop, causing serious damage to the product.
4. Do not overload wall outlets and extension cords as this can result in fire or electric shock.
5. Never push foreign objects of any kind into this product through cabinet slots as they may affect dangerous voltage points or short circuit parts that could result in fire or electric shock. Never spill liquid of any kind on the product.
6. Avoid moving or disconnecting this unit during a heavy storm, as there may be a remote risk of electric shock from lightning and thunder.
7. Unprofessional person cannot open telephone even meets troubles, please send to service center.

DEVICES

- Big number keys designed for elderly
- Hearing-aid compatible (optional)
- Handset receiver booster function
- 8 one-touch memories (including 4 photograph keys)
- Flash and redial function
- Flash time selected with switch
- Earphone and Hands-free volume adjust with potentiometer
- Ringer volume high, low and off switch
- Hands-free dialing and speaking
- Bright ring LED indicator

GUARANTEE

From the moment your Geemarc PhotoPHONE110 is purchased, Geemarc guarantee it for the period of two years. During this time, all repairs or replacements (at our discretion) are free of charge. Should you experience a problem then contact our help line or visit our website at **www.geemarc.com**. The guarantee does not cover accidents, negligence or breakages to any parts. The product must not be tampered with or taken apart by anyone who is not an authorized Geemarc representative. The Geemarc guarantee in no way limits your legal rights.

IMPORTANT: YOUR RECEIPT IS PART OF YOUR GUARANTEE AND MUST BE RETAINED AND PRODUCED IN THE EVENT OF A WARRANTY CLAIM.

**Please note: The guarantee applies to
the United Kingdom only**

DECLARATION: Geemarc Telecom SA hereby declares that this PhotoPhone110 is in compliance with the essential requirements and other relevant provisions of the Radio and Telecommunications Terminal Equipment Directive RED 2014/30/UE.

*The declaration of conformity may be consulted at
www.geemarc.com*

RECYCLING DIRECTIVES

The WEEE (Waste Electrical and Electronic Equipment) has been put in place for the products at the end of their useful life are recycled in the best way.

When this product is finished with, please do not put it in your domestic waste bin.

Please use one of the following disposal options:

- Remove the batteries and deposit them in an appropriate WEEE skip. Deposit the product in an appropriate WEEE skip.
- Or, hand the old product to the retailer. If you purchase a new one, they should accept it.

Thus if you respect these instructions you ensure human health and environmental protection.

For product support and help visit our website at

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This product is made in China to Geemarc Telecom

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